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EN

Field of use

Kering Eyewear S.p.A. optical frames are class I medical devices (Rule 1 Chapter III of Annex VIII of (EU) Regulation 2017/745), which are principally intended for use in connection with ophthalmic prescription lenses for correcting visual problems.

Optical frames comply with (EU) Regulation 2017/745.

Warning

Adverse contact reactions to materials and treatments used for the frames in particularly sensitive or allergic subjects cannot be ruled out.

The frames come with demonstration lenses that are not intended for use but are designed to only prevent frames from warping and allow for a more realistic try-on.

Storage and Cleaning

Store the optical frames in their case at a temperature between -10°C and +35°C.

Keep away from direct sunlight and avoid exposure to high temperatures, such as the heat in a car without air conditioning (e.g. car dashboard), because high temperatures may alter the features of the frames.

In case of contact with agents such as salt water (seawater), swimming pool water, chemical products such as make-up, hairspray, sun lotions, and insect repellents, we recommend that you immediately remove such agents using a soft damp cloth.

In case of wear and tear (e.g. scratched, opaque), we recommend the use of original replacement parts.

Use a damp cloth and mild soap to clean the frames, then dry them with a soft, clean cloth.

Do not use solvents (e.g. alcohol, acetone) or strong detergents which may alter the features of the frames.

Marking

The model identification mark is on the product. The date of manufacture is indicated on the individual packaging.

The information necessary for the traceability of the device is shown on the individual packaging of the product.

Reporting/Contact

In case of complaints or anomalies, should you not be able to contact the store where you purchased the product, please forward your request to: contactus@keringeyewear.com.

READ AND KEEP.